champagner, ginger, champagne 178

nuss, chestnut, noisette 179


melone, melon, melon 181

rioja, aubergine, bordeaux 176

sunset, sunset, mandarine 182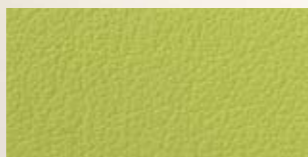

spring, spring, printemps 180


ziegel, amber, rouille 177

pazifik, pacific, pacifique 173

cotto, cotto, cotto 175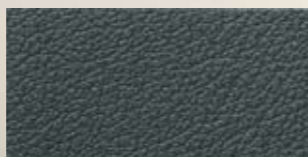

carbon, carbon, carbone 172

DKL

DKL GmbH

An der Ziegelei 4 · D-37124 Rosdorf

Fon +49 (0)551-50 06 0

Fax +49 (0)551-50 06 299

www.neuepolster.de · info@dkl.de

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PLEASE NOTE: We do not accept any liability for discolourations caused by textiles on white, light-coloured, yellow, light green, light blue, beige and orange upholstery parts.

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FashionLINE

Properties:

This material has a leather-like surface with a skin-friendly coating and an interesting structure. It is waterproof and can be disinfected.

Care Instructions:

Never use disinfectants containing alcohol. We recommend applying our spray foam proPad to your upholstery. Clean the upholstery in circular motions with a soft sponge and lukewarm water at the end of each treatment day. Never use cleaning agents or household cleaners. After being cleaned, the upholstery should be wiped dry with a cloth. Then apply DentaProtect (wipe dispenser), our sealant for imitation leather.

Soiling / Discolouration

Light colours such as white, yellow, light green, light blue, beige or orange, are particularly susceptible to soiling or discolouration. Residual humidity from disinfectants enhances the migration of colourants from textiles. Particularly aggressive discolouration stains are caused by belts and suede leather clothes. Carefully wipe the upholstery dry after the residence time of the surface disinfectant! Soilings can be removed with water and a sponge or a brush. Please note: discolourations caused by jeans or other textiles excluded from manufacturer's guarantee.

You will find detailed information on this topic in the DKL care brochure, which is available for download on our website www.neuepolster.de

Please note that, for technical reasons, the colours may differ slightly from the original surfaces.

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